## **3D VIEWER SETTINGS**

3D rotation \*

1 or

Zoom in/out

Click to set the volume rotation center



3D Pan

## **GENERAL SHORTCUTS**

Modellation tool size



Modellation tool strength



Selection area with the brush



Deselection area with the brush



Undo

Z + \* Ctrl / Cmd

Redo

Y + Ctrl / Cmd



## **ONLY CAD**

Change the contact scale



Hide the clicked 3D object \*\*

+ \* F1 / fn + F1

Change the clicked 3D object transparency \*\*



Show the last hidden object \*\*



Rotate tooth



Scale tooth



Stretch tooth



Multiple selection teeth on dentogram



- \* by default; customize it into project Settings/General switching off the "Use Left Mouse [...]" button
- \*\* by default; customize it into project Settings/Shortcut
- It is also possible to use ONLY the function keys (F1, F2, F3) on the Mac-OS system by enabling them. Select the Apple menu > System > Preferences > Select "Keyboard" > Click on "Keyboard" > Enable "Use F1, F2, etc... as standard function keys"

